



## Sinus Lateral Approach Kit



# A Minimally Invasive Approach to Lateral Sinus Lifts

## 1 MINIMALLY INVASIVE

- Small lateral window
- Minimal flap size

## 2 SPEED AND SAFETY


- Designed for 2,000rpm
- Control of drilling depth up to 3.5mm without drill stops (LS Reamer)

## 3 COMPACT AND COMPLETE


- Complete instrumentations for two different surgical techniques
- Customize surgical approach between LS Reamer- or C-Reamer to preserve bone core

# SINUS LATERAL APPROACH KIT TECHNIQUE GUIDE


## TWO PATHS - ONE PRODUCT

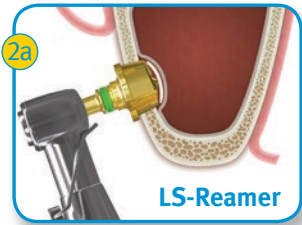


- Make a lateral hole with the LS-Reamer.
- Keep drill perpendicular to the bone and maintain steady drilling pressure.
- Recommended drill speed: up to 2,000rpm.

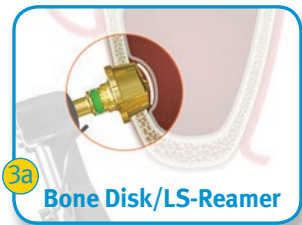


- Drill the lateral wall until a thin bone layer is left and a thin residual bone shield is formed with LS-Reamer.

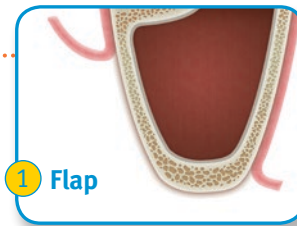

- Elevate small flap.

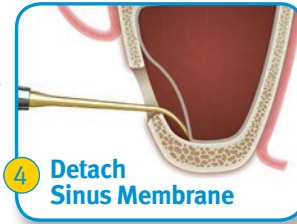

- #1 elevator is used to initiate detachment of the membrane in the mesial-distal area.

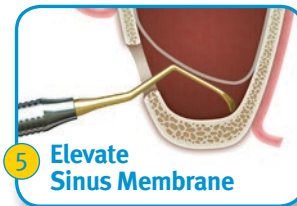


- #2 elevator is used for detaching membrane in the inferior area.
- #3 elevator is used for anterior and posterior areas.




- Place bone graft material.
- Cover the lateral hole with a collagen Membrane (LS-Reamer approach) or the bone disk (C-Reamer approach).
- Suture soft tissue.



- Make a pilot marking with the C-Guide.

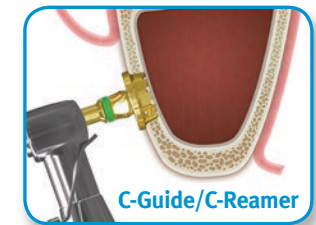

- Make lateral hole while preserving the existing bone core with C-Reamer.
- Recommended drill speed: up to 2,000rpm.

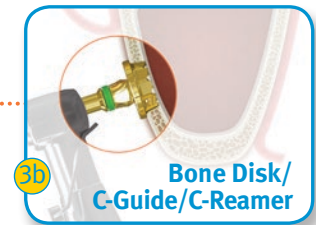

- Drill until circular bone disk is formed from the lateral wall with C-Guide/C-Reamer.

### Note: Use copious irrigation during drilling!

Note: This guide is for educational use only.  
See instructions for use for additional information on this product.

### Ordering information

Catalog Number	Description
SLAKIT	Sinus Lateral Approach Instrument Kit

To learn more about this product, please visit us online at [www.zimmerdental.com](http://www.zimmerdental.com) or to speak to a sales representative, call 1 (800) 854-7019 (in the US) or +1 (760) 929-4300 (outside the US).

To receive our eNews visit us at [http://www.zimmerdental.com/news\\_eNewsletterSignUp.aspx](http://www.zimmerdental.com/news_eNewsletterSignUp.aspx)

For more information about our Products, Regenerative Materials and Educational Opportunities, contact us:

In the U.S. 1 (800) 854-7019  
To fax an order 1 (888) 225-2483  
Outside the U.S. +1 (760) 929-4300  
Australia +61 (0)2 9950 5434 or 1 (800) 241 916  
Canada +1 (905) 567-2073 or 1 (800) 265-0968  
Chile +562 231 5185

China + 86 21 2211 5147  
France +33 (0)1 45 12 35 35  
Germany +49 (0)761 1 56 47 0  
Israel +972 (0)3 6124242  
Italy +39 0438 37681  
Spain +34 93 846 05 43



Zimmer Dental  
1900 Aston Avenue  
Carlsbad, CA 92008-7308  
USA

[www.zimmerdental.com](http://www.zimmerdental.com)