



## **Safescraper® TWIST Cortical Bone Collector**



*A New Twist On Manually Harvesting Bone*

- **Provides 160° Cutting Area – Effective On Plane, Concave And Convex Bone Surfaces**
- **Available In Curved And Straight Designs – Versatile, Facilitating Access To Hard-To-Reach Posterior Regions**
- **Large Capacity Chamber – Minimizes Bone Shaving Dispersion And Retains Bone In A Sterile Chamber To Limit Patient Contamination**
- **Lateral Opening System – Provides Device Stability And May Be Used As A Container For Composite Grafts**
- **Minimally Invasive – The Manual Harvesting Technique Reduces Operating Time And Post-Operative Patient Discomfort**



## Safescraper® TWIST: How It Works

1. The bone is collected by pushing the end of the device toward the bone surface with the chamber facing downward and simultaneously pulling the device itself backward.
2. To retrieve collected bone, keep the transparent chamber facing downward, press the lock button sideways and slide the chamber clockwise.

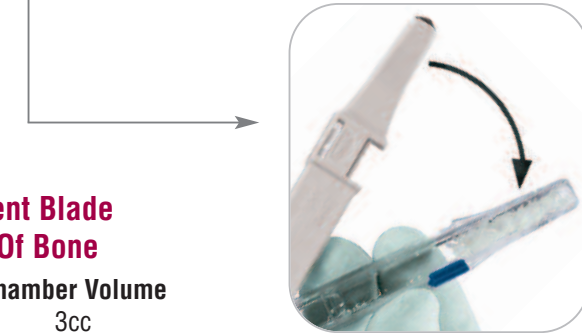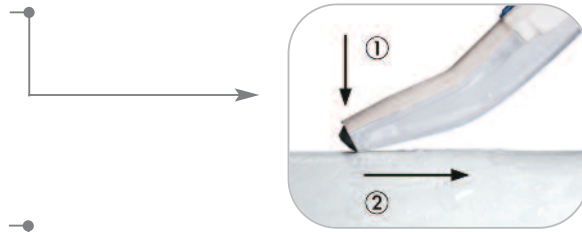

**Each Safescraper TWIST Has A Per-Patient Blade Life That Effectively Harvests Up To 5cc Of Bone**

Ordering Codes	Chamber Volume
3598 Safescraper TWIST Straight - 3 units	3cc
3987 Safescraper TWIST Curve - 3 units	2.5cc

Shelf life of Safescraper TWIST is three years.

Distributed by:



Global Headquarters  
4555 Riverside Drive  
Palm Beach Gardens, FL 33410  
1-800-342-5454  
Outside The U.S.: +1-561-776-6700  
Fax: +1-561-776-1272  
www.biomet3i.com

**Get A New Twist For Your Bone Graft And Defect Cases By Contacting Your BIOMET 3i Sales Representative Today.**

Sign Up For BIOMET 3i's Electronic Newsletter "BIOMET 3innovations." Simply Go Online To [www.biomet3i.com/signup](http://www.biomet3i.com/signup)



INST819  
REV A 06/14

Safescraper is a registered trademark of CGM S.p.A Divisione Medicale META. BIOMET is a registered trademark and BIOMET 3i and design are trademarks of BIOMET, Inc. ©2008 BIOMET 3i, Inc. All rights reserved.

### SUBSIDIARIES

**AUSTRALIA**  
Phone: +61-2-9855-4444  
Fax: +61-2-9888-9900

**BRAZIL**  
Phone: +55-11-5081-4405  
Fax: +55-11-5081-7484

**CANADA**  
Phone: +514-956-9843  
Fax: +514-956-9844

**FRANCE**  
Phone: +33-1-41054343  
Fax: +33-1-41054340

**IRELAND**  
Phone: +35-31-477-3925  
Fax: +35-31-402-9590

**THE NETHERLANDS**  
Phone: +31-(0)78-629-2800  
Fax: +31-(0)78-629-2801

**NEW ZEALAND**  
Phone: +64-508-122-221  
Fax: +64-508-133-331

**NORDIC REGION**  
Phone: +46-40-17-6090  
Fax: +46-40-17-6099

**PORTUGAL**  
Phone: +351-21-000-1645  
Fax: +351-21-000-1675

**SPAIN**  
Phone: +34-93-470-59-50  
Fax: +34-93-372-11-25

**U.K.**  
Phone: +44-1628-829314  
Fax: +44-1628-820182

### DISTRIBUTORS

**ARGENTINA**  
Dentalmax, SA  
Phone: +541-1482-71001  
Fax: +541-1482-67373

**CHILE**  
Cybel, SA  
Phone: +56-2-2321883  
Fax: +56-2-2330176

**CHINA**  
Atek Inc.  
Phone: +86-21-6329-1265  
Fax: +86-21-6329-1620

**COLOMBIA**  
3i Colombia  
Phone: +571-612-9362  
Fax: +571-620-6412

**COSTA RICA**  
Implantec S.A.  
Phone: +506-2-256411  
Fax: +506-2-247620

**EL SALVADOR**  
Dentimerc SA de CV  
Phone: +503-263-6350  
Fax: +503-263-6676

**GREECE**  
Kostas Kornisorlis and Co.  
Phone: +30-2310-501-651  
Fax: +30-2310-522-417

**HONG KONG**  
Ositek Inc., Ltd.  
Phone: +852-8121-6601  
Fax: +852-3747-3754

**PARAGUAY**  
Andres H. Arca y Cia SRL  
Phone: +595-21-208185  
Fax: +595-21-496291

**POLAND**  
Dental Depot  
Phone: +48-71-341-3091  
Fax: +48-71-343-6560

**RUSSIA**  
Com-Dental  
Phone: +7-495-797-6686  
Fax: +7-499-242-9567

**SINGAPORE**  
Asia Implant Support & Services  
Phone: +65-6223-2229  
Fax: +65-6220-3538

**SOUTH AFRICA**  
Selective Surgical CC  
Phone: +27-11-991-7007  
Fax: +27-11-672-1391

**THAILAND**  
3i (Thailand) Co., LTD.  
Phone: +662-252-6685  
Fax: +662-252-6686

**UKRAINE**  
Com-Dental  
Phone: +38-067-7007667  
Fax: +38-044-5017117

**URUGUAY**  
Pro3implant S.R.L.  
Phone: +598-2-4034163  
Fax: +598-2-4034163